

Instructions for Use

Prostate Biopsy Kits



CATALOG NUMBER	DESCRIPTION	UNIT OF MEASUREMENT
PBKPF6	Prostate Biopsy Kits with Prostate Fixative, 6 Vials	1 each
PBKPF12	Prostate Biopsy Kits with Prostate Fixative, 12 Vials	1 each
PBKNB6	Prostate Biopsy Kits with NBF, 6 Vials	1 each
PBK0507	Prostate Biopsy Kits with NBF, 8 Vials	1 each
PBKNB12	Prostate Biopsy Kits with NBF, 12 Vials	1 each
PBKNB16	Prostate Biopsy Kits with NBF, 16 Vials	1 each
PBKNB2/120/CASS	Prostate Biopsy 2 Vial Kit with Cassettes	1 each

INTENDED USE

Prostate Biopsy Kits are used to collect, preserve, and transport prostate tissue samples.

APPLICATIONS

Used to collect multiple biopsy samples for mapping, labeling, and transporting prostate tissue. Each kit includes pre-labeled positions in the box that correlate to the sample area, biopsy location stickers, and prefill containers with an audible clicking sound to eliminate leaks.

Specimen containers contain either 10% NBF or Prostate Fixative.

Prostate Biopsy Kit with Cassettes contains 12 pre-labeled cassettes.

STORAGE AND STABILITY

Storage: Room temperature.
Refer to SDS for details

SPECIMEN PREPARATION

Tissue specimens should be placed in fixative immediately after removal from the body. Prompt fixation will prevent autolysis, putrefaction, and other undesirable cellular changes.

PROCEDURE

- 1 Properly label all specimen containers with biopsy location sticker.
- 2 Place the required patient identification information on the pre-filled container, making sure to use two patient identifiers for positive patient identification.
- 3 After retrieval of tissues, immediately place the specimen into the corresponding pre-labeled container.
- 4 Securely close container, listening for one click. Do not overtighten lid.
- 5 Visually inspect the container to confirm the cap is properly aligned and fully engaged to prevent leaking.
- 6 Place container in pre-labeled position in the box for easy transport.

Please contact productsupport@statlab.com with any additional questions.



StatLab Medical Products
2090 Commerce Drive, McKinney, TX 75069
StatLab.com Phone: +1 800-442-3573