Human Skin

Melanin Bleach Kit Procedure

100ml Kit Item #: KTMBK

Liter Kit Item#: N/A

Pint Kit Item #: N/A

Gallon Kit Item#: N/A

Control Slide(s)	Item#
Skin	CSS0825P

Included Components
1% Potassium Permanganate
2% Oxalic Acid

PRINCIPLE AND RESULTS: This kit is intended for use by laboratory professionals to remove the melanin pigment (in vitro) from hair, skin, retina or substantia nigra of the brain.

SPECIMEN CRITERIA: Appropriately fixed, paraffin-embedded, 4-5µm tissue section.

STORAGE AND USAGE NOTES: Store/Use each component according to the temperature and expiration on the label.

PRECAUTIONS: For use by laboratory professionals. See SDS for complete warnings, precautions, hazard and precautionary statements, and disposal information.

STAINING PROCEDURE:

 Color coordinated steps denote stain baths that can be reused during autostainer configuration.

#	Action	With	Heat °C	Time		Details
				Mins	Secs	
1	Deparaffinize	Xylene or Substitute, 2 changes	--	5	--	5 minutes each change or as required if using a xylene substitute.
2	Rinse	Absolute Alcohol, 3 changes	--	1	--	1 minute each change or as required if using graded alcohols.
3	Rinse 	Running DI Water	--	1	--	
4	Immerse	1% Potassium Permanganate	--	20-240	--	Adjust time depending on tissue type. Once complete, rinse in running DI water (1 minute) and continue.
5	Immerse	2% Oxalic Acid	--	1-2	--	Until tissue is colorless. Once complete, rinse in running DI water (1 minute) . Perform special or IHC stain per instructions excluding deparaffinization and hydration steps.

1. Lillie R: Histopathologic Technique and Practical Histochemistry 3rd Edition, 1965
2. With modifications by AMTS R&D Department, 1979-2018.

AUTOSTAINER CONFIGURATION AND NOTES:

This stain kit is not available in the pint or larger size.

TEST YIELD: *Assumes pint kit and maximum slides per run. Actual Results may vary. S.C. denotes number of slides between "Solution Change".

Bath Type	Uses	Slides	S.C.	Bath Type	Uses	Slides	S.C.
20ml Plastic Slide Jar	5	10	20	250ml Glass Stain Dish	n/a	n/a	n/a
30ml Glass Coplin Jar	3	90	30	200ml Bath Autostainer	n/a	n/a	n/a
40ml Hellendahl Jar	2	80	40	400ml Bath Autostainer	n/a	n/a	n/a

CE MARKINGS AND DESIGNATIONS:

REF	Catalogue Number		Temperature Limitation		Manufacturer	StatLab Medical Products 2090 Commerce Drive McKinney, TX 75069 www.statlab.com 800-442-3573
LOT	Batch Code		Use By			
IVD	In Vitro Diagnostic Medical Device		Consult Instructions Prior to Use			

1. Lillie R: Histopathologic Technique and Practical Histochemistry 3rd Edition, 1965
2. With modifications by AMTS R&D Department, 1979-2018.

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COMPOSANTS INCLUS: 1% Potassium Permanganate, 2% Oxalic Acid

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