Normal Human Colon

Muci Plus™ Stain Kit Procedure

100ml Kit Item#: KTMLP

Liter Kit Item#: N/A

Pint Kit Item#: KTMLPPT

Gallon Kit Item#: N/A

Control Slide(s)
Item#

Colon CSC1025P

Small Intestine CSS0325P

Included Components

Muci-10 Plus™

Modified Mayer's Hematoxylin

Metanil Yellow Stain

PRINCIPLE AND RESULTS: This kit is intended for use by laboratory professionals to stain routinely prepared paraffin embedded tissue specimens (in vitro) to identify epithelial mucins. Epithelial mucins stain pink to red and nuclei blue.

SPECIMEN CRITERIA: Appropriately fixed, paraffin-embedded, 4-5µm tissue section.

STORAGE AND USAGE NOTES: Store/Use each component according to the temperature and expiration on the label.

PRECAUTIONS: For use by laboratory professionals. See SDS for complete warnings, precautions, hazard and precautionary statements, and disposal information.

MUCI-10 PLUS™ WORKING SOLUTION PREPARATION: Prepare solution at time of use. Solution expires after one use.

#	Action	Amount	Chemical/Reagent	Details
1	Add	90ml	Distilled/DI Water	Into a chemically cleaned container or new/unused plasticware.
2	Add	10ml	Muci-10 Plus™	Mix thoroughly.

STAINING PROCEDURE:

 Color coordinated steps denote stain baths that can be reused during autostainer configuration.

#	Action	With	Heat °C	Time		Details
				Mins	Secs	
1	Deparaffinize	Xylene or Substitute, 2 changes	--	5	--	5 minutes each change or as required if using a xylene substitute.
2	Rinse	Absolute Alcohol, 3 changes	--	1	--	1 minute each change or as required if using graded alcohols.
3	Rinse	Running Tap Water	--	1	--	
4	Immerse	Modified Mayer's Hematoxylin	--	5	--	Once complete, rinse in running tap water (1 minute) and continue.
5	Immerse	Muci-10 Plus™ Working Solution	--	10	--	If deeper color is desired, extend staining time. Once complete, rinse in running tap water (5 seconds) and continue.
6	Immerse	Metanil Yellow Stain	--	--	15-60	Once complete, rinse in running tap water (5 seconds) and continue.
7	Dehydrate	Absolute Alcohol, 3 changes	--	1	--	1 minute each change.
8	Clear	Xylene or Substitute, 3 changes	--	1	--	1 minute each change or as required if using a xylene substitute.
9	Coverslip	Permanent Mounting Media	--	--	--	

1. Sheehan DC Hrapchak BB: Theory and Practice of Histotechnology, 1980, 168 – 169
2. With modifications by AMTS R&D Department, 1979-2019.

AUTOSTAINER CONFIGURATION AND NOTES:

This stain kit in the pint or larger size may be easily adapted for use on most open-platform autostainers using the staining procedure grid on the reverse side of this page. Standard Muci-10 Plus™ Working Solution preparation procedure yields approximately 100ml of solution and must be scaled up to accommodate desired autostainer bath size. A minimum of 4 baths is required to perform this procedure excluding deparaffinization, hydration, dehydration, and clearing, or 15 baths to run the complete procedure.

TEST YIELD: *Assumes pint kit and maximum slides per run. Actual Results may vary. S.C. denotes number of slides between "Solution Change".

Bath Type	Uses	Slides	S.C.	Bath Type	Uses	Slides	S.C.
20ml Plastic Slide Jar	25	500	20	250ml Glass Stain Dish	2	375	188
30ml Glass Coplin Jar	16	480	30	200ml Bath Autostainer	2	320	160
40ml Hellendahl Jar	12	480	40	400ml Bath Autostainer	1	340	340

CE MARKINGS AND DESIGNATIONS:

REF	Catalogue Number		Temperature Limitation		Manufacturer	American MasterTech Scientific 1330 Thurman St. Lodi, CA 95240 USA Tel 800 860 4073 Fax 209 368 4136
LOT	Batch Code		Use By	EC REP	Representative	Emergo Europe Prinsessegracht 20 2514 AP The Hague The Netherlands
IVD	In Vitro Diagnostic Medical Device		Consult Instructions Prior to Use	 GHS08	Health Hazard	CE

CONTACT INFORMATION:

American MasterTech Scientific
 1330 Thurman St.
 Lodi, CA 95240 USA
 Tel 800 860 4073
 Fax 209 368 4136
www.americanmastertech.com

StatLab
 2090 Commerce Drive
 McKinney, TX 75069 USA
 Tel 800 442 3573
 Fax 972 436 1369
www.statlab.com

1. Sheehan DC Hrapchak BB: Theory and Practice of Histotechnology, 1980, 168 – 169
2. With modifications by AMTS R&D Department, 1979-2019.

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COMPOSANTS INCLUS: Muci-10 Plus™, Modified Mayer's Hematoxylin, Metanil Yellow Stain

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