Human Uterus

Trichrome One-Step Blue & Red Stain Kit Procedure

100ml Kit Item #: N/A

Liter Kit Item#: KTTRBLT

Pint Kit Item #: KTTRBPT

Gallon Kit Item#: KTTRBGAL

Control Slide(s)
Item#

Heart	CSH0725P
Skeletal Muscle	CSS0725P
Trichrome	CST0125P
Uterus	CSU0325P

Included Components

Bouin's Fluid
Modified Mayer's Hematoxylin
One-Step Trichrome Stain (Blue & Red)

PRINCIPLE AND RESULTS: This kit is intended for use by laboratory professionals to stain routinely prepared paraffin embedded tissue specimens (in vitro) to identify collagen and muscle. Cytoplasm, keratin, muscle, intercellular fibers are stained red, collagen and mucus blue, and nuclei blue.

SPECIMEN CRITERIA AND CONTROL TISSUE: Appropriately fixed, paraffin-embedded, 4-6µm tissue section.

STORAGE AND USAGE NOTES: Store/Use each component according to the temperature and expiration on the label.

PRECAUTIONS: For use by laboratory professionals. See SDS for complete warnings, precautions, hazard and precautionary statements, and disposal information.

STAINING PROCEDURE: Preheating Required. See step 4 for more information.

 Color coordinated steps denote stain baths that can be reused during autostainer configuration.

#	Action	With	Heat °C	Time		Details
				Mins	Secs	
1	Deparaffinize	Xylene or Substitute, 2 changes	--	5	--	5 minutes each change or as required if using a xylene substitute.
2	Rinse	Absolute Alcohol, 3 changes	--	1	--	1 minute each change or as required if using graded alcohols.
3	Rinse	Running Tap Water	--	1	--	
4	Waterbath	Bouin's Fluid	56°	60	--	Immerse into preheated solution. Mordant in Bouin's Fluid. For enhanced results, mordant overnight at room temperature. Once complete, rinse in running tap water (3 minutes) or until tissue is colorless and continue.
5	Immerse	Modified Mayer's Hematoxylin	--	7	--	Once complete, rinse in running tap water (3 minutes) and continue.
6	Immerse	One Step Trichrome Stain (Blue & Red)	--	3-8	--	Once complete, rinse in running tap water (5 seconds) and continue. <i>NOTE: If tissue is consistently too blue, reduce stain time or differentiate colors by immersing in a combined 1% Acetic Acid and 0.7% Phosphotungstic Acid Solution for 5-10 seconds.</i>
7	Dehydrate	Absolute Alcohol, 3 changes	--	1	--	1 minute each change.
8	Clear	Xylene or Substitute, 3 changes	--	1	--	1 minute each change or as required if using a xylene substitute.
9	Coverslip	Permanent Mounting Media	--	--	--	

1. Sheehan DC Hrapchak BB: Theory and Practice of Histotechnology; 1980, 191 - 192.
2. Luna LG: Histopathologic Methods and Color Atlas of Special Stains and Tissue Artifacts; 1992, 418 - 419.
3. With modifications by AMTS R&D Department, 1979-2019.

AUTOSTAINER CONFIGURATION AND NOTES:

This stain kit in the pint or larger size may be easily adapted for use on most open-platform autostainers using the staining procedure grid on the reverse side of this page. A minimum of 4 baths is required to perform this procedure excluding deparaffinization, hydration, dehydration, and clearing, or 15 baths to run the complete procedure.

TEST YIELD: *Assumes pint kit and maximum slides per run. Actual Results may vary. S.C. denotes number of slides between "Solution Change".

Bath Type	Uses	Slides	S.C.	Bath Type	Uses	Slides	S.C.
20ml Plastic Slide Jar	25	500	20	250ml Glass Stain Dish	2	375	188
30ml Glass Coplin Jar	16	480	30	200ml Bath Autostainer	2	320	160
40ml Hellendahl Jar	12	480	40	400ml Bath Autostainer	1	340	340

CE MARKINGS AND DESIGNATIONS:

REF	Catalogue Number		Temperature Limitation		Manufacturer	American MasterTech Scientific 1330 Thurman St. Lodi, CA 95240 USA Tel 800 860 4073 Fax 209 368 4136
LOT	Batch Code		Use By	EC REP	Representative	Emergo Europe Prinsessegracht 20 2514 AP The Hague The Netherlands
IVD	In Vitro Diagnostic Medical Device		Consult Instructions Prior to Use			
 GHS05	Corrosive	 GHS07	Irritant	 GHS08	Health Hazard	

CONTACT INFORMATION:

American MasterTech Scientific
 1330 Thurman St.
 Lodi, CA 95240 USA
 Tel 800 860 4073
 Fax 209 368 4136
www.americanmastertech.com

StatLab
 2090 Commerce Drive
 McKinney, TX 75069 USA
 Tel 800 442 3573
 Fax 972 436 1369
www.statlab.com

1. Sheehan DC Hrapchak BB: Theory and Practice of Histotechnology; 1980, 191 - 192.
2. Luna LG: Histopathologic Methods and Color Atlas of Special Stains and Tissue Artifacts; 1992, 418 - 419.
3. With modifications by AMTS R&D Department, 1979-2019.

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COMPOSANTS INCLUS: Bouin's Fluid, Modified Mayer's Hematoxylin, One-Step Trichrome Stain (Bleu et Rouge)

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COMPONENTES INCLUIDOS: Bouin's Fluid, Modified Mayer's Hematoxylin, One-Step Trichrome Stain (Blue & Red)

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