Human Spleen

## Advanced Carbol Fuchsin™ Stain Procedure

**100ml Item #:** STACF100

**Liter Item#:** STACFLT

**Pint Item #:** STACFPT

**Gallon Item#:** STACFGAL

| Control Slide(s)   | Item#    |
|--------------------|----------|
| Acid Fast Bacteria | CSA0125P |

**Required Components (Not Included)**

 1% Acid Alcohol  
 Light Green Counterstain or Methylene Blue Stain

**PRINCIPLE AND RESULTS:** This stain is intended for use by laboratory professionals to stain routinely prepared paraffin embedded tissue specimens (in vitro) to identify Acid Fast Bacteria (mycobacteria), including *Mycobacterium tuberculosis* and *Mycobacterium avium intracellulare*. All Acid Fast Bacteria stain bright red and other tissue green or blue (depending on what counterstain is chosen). Compared to the Carbol Fuchsin solution used in Kinyoun's AFB and Ziehl-Neelsen procedures, MasterTech Advanced Carbol Fuchsin™ requires no heat and has a much shorter staining time by combining all three fuchsin dyes.

**SPECIMEN CRITERIA:** Appropriately fixed, paraffin-embedded, 4-5µm tissue section.

**STORAGE AND USAGE NOTES:** Filter Advanced Carbol Fuchsin™ when a shiny gold layer appears on the surface. Store/Use component according to the temperature and expiration on the label.

**PRECAUTIONS:** For use by laboratory professionals. See SDS for complete warnings, precautions, hazard and precautionary statements, and disposal information.

### STAINING PROCEDURE:

 Color coordinated steps denote stain baths that can be reused during autostainer configuration.

| #                              | Action                                                                                        | With                            | Heat °C | Time  |       | Details                                                                                                                                                                                                                |
|--------------------------------|-----------------------------------------------------------------------------------------------|---------------------------------|---------|-------|-------|------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
|                                |                                                                                               |                                 |         | Mins  | Secs  |                                                                                                                                                                                                                        |
| 1                              | Deparaffinize                                                                                 | Xylene or Substitute, 2 changes | --      | 5     | --    | 5 minutes each change or as required if using a xylene substitute.                                                                                                                                                     |
| 2                              | Rinse                                                                                         | Absolute Alcohol, 3 changes     | --      | 1     | --    | 1 minute each change or as required if using graded alcohols.                                                                                                                                                          |
| 3                              | Rinse                                                                                         | Running Tap Water               | --      | 1     | --    |                                                                                                                                                                                                                        |
| 4                              | Immerse                                                                                       | Advanced Carbol Fuchsin™        | --      | 10-15 | --    | Once complete, <b>rinse in running tap water (1 minute)</b> and continue.                                                                                                                                              |
| 5                              | Immerse                                                                                       | 1% Acid Alcohol                 | --      | --    | 3-5   | Differentiate with agitation until background is pale pink. Use a quick tap water rinse and microscope to check for correct differentiation. Once complete, <b>rinse in running tap water (1 minute)</b> and continue. |
| 6                              | Choose counterstain below. For Light Green follow step 6a. For Methylene Blue follow step 6b. |                                 |         |       |       |                                                                                                                                                                                                                        |
| 6a Light Green Counterstain    |                                                                                               |                                 |         |       |       |                                                                                                                                                                                                                        |
|                                | Immerse                                                                                       | Light Green Counterstain        | --      | --    | 30-60 | Without rinsing, continue to next step.                                                                                                                                                                                |
|                                | Dehydrate                                                                                     | Absolute Alcohol, 3 changes     | --      | 1     | --    | 1 minute each change. Once complete, continue to step 7.                                                                                                                                                               |
| 6b Methylene Blue Counterstain |                                                                                               |                                 |         |       |       |                                                                                                                                                                                                                        |
|                                | Immerse                                                                                       | Methylene Blue Counterstain     | --      | --    | 1-5   | Once complete, <b>rinse in running tap water (10-15 seconds)</b> to remove excess stain.                                                                                                                               |
|                                | Dehydrate                                                                                     | Absolute Alcohol, 2 changes     | --      | --    | 5     | 5 seconds each change. Once complete, continue to step 7.                                                                                                                                                              |
| 7                              | Clear                                                                                         | Xylene or Substitute, 3 changes | --      | 1     | --    | 1 minute each change or as required if using a xylene substitute. Use fresh xylene if Methylene Blue was used as a counterstain.                                                                                       |
| 8                              | Coverslip                                                                                     | Permanent Mounting Media        | --      | --    | --    |                                                                                                                                                                                                                        |

1. Sheehan DC Hrapchak BB: Theory and Practice of Histotechnology, 1980, 235-239
2. With modifications by AMTS R&D Department, 1979-2019.

**CE MARKINGS AND DESIGNATIONS:**

|                                                                                            |                                    |                                                                                            |                                   |                                                                                   |                |                                                                                                                  |
|--------------------------------------------------------------------------------------------|------------------------------------|--------------------------------------------------------------------------------------------|-----------------------------------|-----------------------------------------------------------------------------------|----------------|------------------------------------------------------------------------------------------------------------------|
| <b>REF</b>                                                                                 | Catalogue Number                   |           | Temperature Limitation            |  | Manufacturer   | American MasterTech Scientific<br>1330 Thurman St.<br>Lodi, CA 95240 USA<br>Tel 800 860 4073<br>Fax 209 368 4136 |
| <b>LOT</b>                                                                                 | Batch Code                         |           | Use By                            | <b>EC REP</b>                                                                     | Representative | Emergo Europe<br>Prinsessegracht 20<br>2514 AP The Hague<br>The Netherlands                                      |
| <b>IVD</b>                                                                                 | In Vitro Diagnostic Medical Device |           | Consult Instructions Prior to Use | <b>CE</b>                                                                         |                |                                                                                                                  |
| <br>GHS05 | Corrosive                          | <br>GHS08 | Health Hazard                     |                                                                                   |                |                                                                                                                  |

**CONTACT INFORMATION:**

**American MasterTech Scientific**  
 1330 Thurman St.  
 Lodi, CA 95240 USA  
 Tel 800 860 4073  
 Fax 209 368 4136  
[www.americanmastertech.com](http://www.americanmastertech.com)

**StatLab**  
 2090 Commerce Drive  
 McKinney, TX 75069 USA  
 Tel 800 442 3573  
 Fax 972 436 1369  
[www.statlab.com](http://www.statlab.com)

1. Sheehan DC Hrapchak BB: Theory and Practice of Histotechnology, 1980, 235-239
2. With modifications by AMTS R&D Department, 1979-2019.

## MULTILANGUAGE PROCEDURE

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**COMPOSANTS NON INCLUS:** 1% Acid Alcohol, Light Green Counterstain ou Methylene Blue Counterstain

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