Human Adrenal Gland

## Crystal Violet Amyloid Stain Procedure

0:02

**100ml Item #:** STCVA100

**Liter Item#:** N/A

**Pint Item #:** STCVAPT

**Gallon Item#:** N/A

**Control Slide(s)**
**Item#**

Amyloid

CSA0425P

**Required Components (Not Included)**

Glycerin

Glacial Acetic Acid

**PRINCIPLE AND RESULTS:** This stain is intended for use by laboratory professionals to stain routinely prepared paraffin embedded tissue specimens or frozen sections (in vitro) to identify amyloid. Amyloid stains dark violet.

**SPECIMEN CRITERIA:** Appropriately fixed, paraffin-embedded tissue section cut at 6µm.

**STORAGE AND USAGE NOTES:** Store/Use component according to the temperature and expiration on the label.

**PRECAUTIONS:** For use by laboratory professionals. See SDS for complete warnings, precautions, hazard and precautionary statements, and disposal information.

**1% ACETIC ACID PREPARATION:** For permanent coverslipping staining procedure only! Prepare solution at time of use. Solution expires after one use.

| # | Action | Amount | Chemical/Reagent    | Details                                                        |
|---|--------|--------|---------------------|----------------------------------------------------------------|
| 1 | Add    | 99ml   | Distilled Water     | Into a chemically cleaned container or new/unused plasticware. |
| 2 | Add    | 1ml    | Glacial Acetic Acid | Mix thoroughly.                                                |

### GLYCERIN COVERSLEPPING STAINING PROCEDURE:

 Color coordinated steps denote stain baths that can be reused during autostainer configuration.

| # | Action        | With                            | Heat °C | Time |      | Details                                                                    |
|---|---------------|---------------------------------|---------|------|------|----------------------------------------------------------------------------|
|   |               |                                 |         | Mins | Secs |                                                                            |
| 1 | Deparaffinize | Xylene or Substitute, 2 changes | --      | 5    | --   | 5 minutes each change or as required if using a xylene substitute.         |
| 2 | Rinse         | Absolute Alcohol, 3 changes     | --      | 1    | --   | 1 minute each change or as required if using graded alcohols.              |
| 3 | Rinse         | Running Tap Water               | --      | 1    | --   |                                                                            |
| 4 | Immerse       | Crystal Violet Amyloid Stain    | --      | 2    | --   | Once complete, <b>rinse in running tap water (2 minutes)</b> and continue. |
| 5 | Coverslip     | Glycerin                        | --      | --   | --   | Once complete, have a pathologist read slide immediately.                  |

### PERMANENT COVERSLEPPING STAINING PROCEDURE:

 Color coordinated steps denote stain baths that can be reused during autostainer configuration.

| # | Action        | With                            | Heat °C | Time |       | Details                                                                                                                                                                                                        |
|---|---------------|---------------------------------|---------|------|-------|----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
|   |               |                                 |         | Mins | Secs  |                                                                                                                                                                                                                |
| 1 | Deparaffinize | Xylene or Substitute, 2 changes | --      | 5    | --    | 5 minutes each change or as required if using a xylene substitute.                                                                                                                                             |
| 2 | Rinse         | Absolute Alcohol, 3 changes     | --      | 1    | --    | 1 minute each change or as required if using graded alcohols.                                                                                                                                                  |
| 3 | Rinse         | Running Tap Water               | --      | 1    | --    |                                                                                                                                                                                                                |
| 4 | Immerse       | Crystal Violet Amyloid Stain    | --      | 10   | --    | Once complete, <b>rinse in running tap water (3-5 minutes)</b> and continue.                                                                                                                                   |
| 5 | Immerse       | 1% Acetic Acid                  | --      | --   | 15-20 | Differentiate until proper contrast is achieved. Use a quick tap water rinse and microscope to check for correct differentiation. Once complete, <b>rinse in running tap water (1-2 minutes)</b> and continue. |
| 6 | Air Dry       | On Benchtop, or in Empty Bath   | --      | 1-2  | --    | Until section is completely dry.                                                                                                                                                                               |
| 7 | Clear         | Xylene or Substitute            | --      | --   | 1     | 5-10 dips for 1 second each.                                                                                                                                                                                   |
| 8 | Coverslip     | Permanent Mounting Media        | --      | --   | --    |                                                                                                                                                                                                                |

1. Sheehan DC Hrapchak BB: Theory and Practice of Histotechnology; 1980, 177.
2. With modifications by AMTS R&D Department, 1979-2019.

**CE MARKINGS AND DESIGNATIONS:**

|                                                                                            |                                    |                                                                                            |                                   |                                                                                   |                |                                                                                                                  |
|--------------------------------------------------------------------------------------------|------------------------------------|--------------------------------------------------------------------------------------------|-----------------------------------|-----------------------------------------------------------------------------------|----------------|------------------------------------------------------------------------------------------------------------------|
| <b>REF</b>                                                                                 | Catalogue Number                   |           | Temperature Limitation            |  | Manufacturer   | American MasterTech Scientific<br>1330 Thurman St.<br>Lodi, CA 95240 USA<br>Tel 800 860 4073<br>Fax 209 368 4136 |
| <b>LOT</b>                                                                                 | Batch Code                         |           | Use By                            | <b>EC REP</b>                                                                     | Representative | Emergo Europe<br>Prinsessegracht 20<br>2514 AP The Hague<br>The Netherlands                                      |
| <b>IVD</b>                                                                                 | In Vitro Diagnostic Medical Device |           | Consult Instructions Prior to Use | <b>CE</b>                                                                         |                |                                                                                                                  |
| <br>GHS07 | Irritant                           | <br>GHS08 | Health Hazard                     |                                                                                   |                |                                                                                                                  |

**CONTACT INFORMATION:**

**American MasterTech Scientific**  
 1330 Thurman St.  
 Lodi, CA 95240 USA  
 Tel 800 860 4073  
 Fax 209 368 4136  
[www.americanmastertech.com](http://www.americanmastertech.com)

**StatLab**  
 2090 Commerce Drive  
 McKinney, TX 75069 USA  
 Tel 800 442 3573  
 Fax 972 436 1369  
[www.statlab.com](http://www.statlab.com)

1. Sheehan DC Hrapchak BB: Theory and Practice of Histotechnology; 1980, 177.
2. With modifications by AMTS R&D Department, 1979-2019.

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