Human Uterus

Eosin Y Stain Procedure

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Control Slide(s)	Item#	Required Components (Not Included)	
Trichrome	CST0125P	Harris Hematoxylin	Bluing Solution
Skeletal Muscle	CSS0725P	Differentiating Solution	70% Reagent Alcohol
Smooth Muscle	CSS0425P		

PRINCIPLE AND RESULTS: This stain is intended for use by laboratory professionals as a counterstain for routinely prepared paraffin embedded tissue specimens (in vitro) stained using a hematoxylin and eosin procedure. Cytoplasm and collagen stain shades of pink, muscle stains shades of red, and red blood cells stain red. Made from Biological Stain Commission (BSC) certified dyes, MasterTech Eosin Y has remained unchallenged by the competition for more than 35 years.

SPECIMEN CRITERIA: Appropriately fixed, paraffin-embedded, 4-5µm tissue section.

STORAGE AND USAGE NOTES: Store/Use each component according to the temperature and expiration on the label.

PRECAUTIONS: For use by laboratory professionals. See SDS for complete warnings, precautions, hazard and precautionary statements, and disposal information.

STAINING PROCEDURE:

 Color coordinated steps denote stain baths that can be reused during autostainer configuration.

#	Action	With	Heat °C	Time		Details
				Mins	Secs	
1	Deparaffinize	Xylene or Substitute, 2 changes	--	5	--	5 minutes each change or as required if using a xylene substitute.
2	Rinse	Absolute Alcohol, 3 changes	--	1	--	1 minute each change or as required if using graded alcohols.
3	Rinse	Running Tap Water	--	1	--	
4	Immerse	Harris Hematoxylin	--	5	--	Once complete, rinse is running tap water (1 minute) and continue.
5	Immerse	Differentiating Solution	--	--	1	2-3 dips for 1 second each. Once complete, rinse is running tap water (1 minute) and continue.
6	Immerse	Bluing Solution	--	--	30-40	Once complete, rinse is running tap water (1 minute) and continue.
7	Immerse	70% Reagent Alcohol	--	1	--	Agitate.
8	Immerse	Eosin Y	--	--	30-60	Agitate.
9	Dehydrate	Absolute Alcohol, 3 changes	--	1	--	1 minute each change.
10	Clear	Xylene or Substitute, 3 changes	--	1	--	1 minute each change or as required if using a xylene substitute.
11	Coverslip	Permanent Mounting Media	--	--	--	

1. Sheehan DC Hrapchak BB: Theory and Practice of Histotechnology; 1980, 250 - 251.
2. With modifications by AMTS R&D Department, 1979-2019.

CE MARKINGS AND DESIGNATIONS:

REF	Catalogue Number		Temperature Limitation		Manufacturer	American MasterTech Scientific 1330 Thurman St. Lodi, CA 95240 USA Tel 800 860 4073 Fax 209 368 4136
LOT	Batch Code		Use By	EC REP	Representative	Emergo Europe Prinsessegracht 20 2514 AP The Hague The Netherlands
IVD	In Vitro Diagnostic Medical Device		Consult Instructions Prior to Use	CE		
	Flammable		Health Hazard			
GHS02		GHS08				

CONTACT INFORMATION:

American MasterTech Scientific
 1330 Thurman St.
 Lodi, CA 95240 USA
 Tel 800 860 4073
 Fax 209 368 4136
www.americanmastertech.com

StatLab
 2090 Commerce Drive
 McKinney, TX 75069 USA
 Tel 800 442 3573
 Fax 972 436 1369
www.statlab.com

1. Sheehan DC Hrapchak BB: Theory and Practice of Histotechnology; 1980, 250 - 251.
2. With modifications by AMTS R&D Department, 1979-2019.

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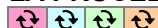
COMPONENTES INCLUIDOS: Harris Hematoxylin, Differentiating Solution, Bluing Solution, Eosin Y Stain

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