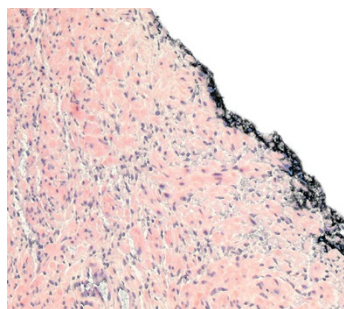
Human Skin

India Ink Stain Procedure

25ml Item#: STIIN25

Pint Item#: STIINPT

0:01

Control Slide(s)	Item#	Required Components (Not Included)
N/A		N/A

PRINCIPLE AND RESULTS: This stain is intended for use by laboratory professionals to stain (in vitro) *Cryptococcus neoformans* in cerebral spinal fluid, which will appear as encapsulated cells that appear to have a clear, unstained zone surrounding the cell. India Ink is also used to mark margins on unfixed tissue specimens.

SPECIMEN CRITERIA: Specimens should be collected in sterile containers and transported to the laboratory immediately.

STORAGE AND USAGE NOTES: Store/Use each component according to the temperature and expiration on the label.

PRECAUTIONS: For use by laboratory professionals. See SDS for complete warnings, precautions, hazard and precautionary statements, and disposal information

STAINING PROCEDURE:

#	Action	With	Heat °C	Time		Details
				Mins	Secs	
1	Add	Cerebral Spinal Fluid	--	--	--	Add one drop to slide.
2	Add	India Ink Stain	--	--	--	Add one drop to slide.
3	Coverslip	Coverslip	--	--	--	Review with a microscope.

CE MARKINGS AND DESIGNATIONS:

REF	Catalogue Number		Temperature Limitation		Manufacturer	American MasterTech Scientific 1330 Thurman St. Lodi, CA 95240 USA Tel 800 860 4073 Fax 209 368 4136
LOT	Batch Code		Use By	EC REP	Representative	Emergo Europe Prinsessegracht 20 2514 AP The Hague The Netherlands
IVD	In Vitro Diagnostic Medical Device		Consult Instructions Prior to Use		Health Hazard	CE

CONTACT INFORMATION:

American MasterTech Scientific
 1330 Thurman St.
 Lodi, CA 95240 USA
 Tel 800 860 4073
 Fax 209 368 4136
 www.americanmastertech.com

StatLab
 2090 Commerce Drive
 McKinney, TX 75069 USA
 Tel 800 442 3573
 Fax 972 436 1369
 www.statlab.com

1. With modifications by AMTS R&D Department, 1979-2019.

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