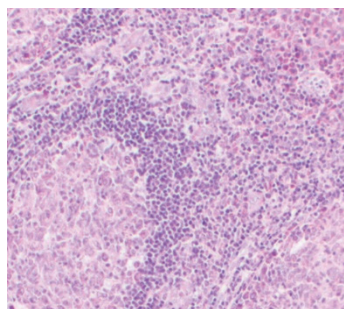
Human Tonsil

Methyl Green Stain Procedure

0:05

100 Item#: STMGP100

Liter Item#: N/A

Pint Item#: STMGPPT

Gallon Item#: N/A

Control Slide(s)
Item#
Required Components (Not Included)

Tonsil

CST0225P

N/A

Mast Cell

CSM0125P

PRINCIPLE AND RESULTS: This stain is intended for use by laboratory professionals to stain routinely prepared paraffin embedded tissue specimens (in vitro) to identify DNA, RNA and Mast Cell Granules. DNA stains blue-green to green, RNA pink to red, and mast cell granules pink.

SPECIMEN CRITERIA: Appropriately fixed, paraffin-embedded, 5µm tissue section.

STORAGE AND USAGE NOTES: Store/Use each component according to the temperature and expiration on the label.

PRECAUTIONS: For use by laboratory professionals. See SDS for complete warnings, precautions, hazard and precautionary statements, and disposal information.

STAINING PROCEDURE:

 Color coordinated steps denote stain baths that can be reused during autostainer configuration.

#	Action	With	Heat °C	Time		Details
				Mins	Secs	
1	Deparaffinize	Xylene or Substitute, 2 changes	--	5	--	5 minutes each change or as required if using a xylene substitute.
2	Rinse	Absolute Alcohol, 3 changes	--	1	--	1 minute each change or as required if using graded alcohols.
3	Rinse	Running Tap Water	--	1	--	
4	Rinse	Running DI Water	--	1	--	
5	Immerse	Methyl Green Pyronin Stain	--	2-7	--	Increasing stain time intensifies Pyronin (red) and decreasing stain time intensifies Methyl Green (blue-green to green). Once complete, rinse in DI water (2 changes, 1-2 dips each change) and continue.
6	Dehydrate	Absolute Alcohol, 3 changes	--	1	--	Fresh changes. 1 minute each change.
7	Clear	Xylene, 3 changes	--	1	--	1 minute each change. Do not use xylene substitute!
8	Coverslip	Permanent Mounting Media	--	--	--	

1. Luna LG: Histopathologic Methods and Color Atlas of Special Stains and Tissue Artifacts; American Histolabs Inc; 1992, 261.
2. With modifications by AMTS R&D Department, 1979-2019.

CE MARKINGS AND DESIGNATIONS:

REF	Catalogue Number		Temperature Limitation		Manufacturer	American MasterTech Scientific 1330 Thurman St. Lodi, CA 95240 USA Tel 800 860 4073 Fax 209 368 4136
LOT	Batch Code		Use By	EC REP	Representative	Emergo Europe Prinsessegracht 20 2514 AP The Hague The Netherlands
IVD	In Vitro Diagnostic Medical Device		Consult Instructions Prior to Use	 GHS08	Health Hazard	CE

CONTACT INFORMATION:

American MasterTech Scientific
 1330 Thurman St.
 Lodi, CA 95240 USA
 Tel 800 860 4073
 Fax 209 368 4136
www.americanmastertech.com

StatLab
 2090 Commerce Drive
 McKinney, TX 75069 USA
 Tel 800 442 3573
 Fax 972 436 1369
www.statlab.com

1. Luna LG: Histopathologic Methods and Color Atlas of Special Stains and Tissue Artifacts; American Histolabs Inc; 1992, 261.
2. With modifications by AMTS R&D Department, 1979-2019.

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