Normal Human Colon

Metanil Yellow Stain Procedure

100ml Item#: STMYE100

Liter Item#: STMYELT

Pint Item#: STMYEPT

Gallon Item#: STMYEGAL

Control Slide(s)	Item#
Colon	CSC1025P
Small Intestine	CSS0325P

Required Components (Not Included)

 Muci-10 Plus™
 Modified Mayer's Hematoxylin

PRINCIPLE AND RESULTS: This stain is intended for use by laboratory professionals to stain routinely prepared paraffin embedded tissue specimens (in vitro) to aid in identifying epithelial mucins by staining background tissue shades of yellow when used in a mucicarmine stain procedure.

SPECIMEN CRITERIA: Appropriately fixed, paraffin-embedded, 4-5µm tissue section.

STORAGE AND USAGE NOTES: Store/Use each component according to the temperature and expiration on the label.

PRECAUTIONS: For use by laboratory professionals. See SDS for complete warnings, precautions, hazard and precautionary statements, and disposal information.

MUCI-10 PLUS™ WORKING SOLUTION PREPARATION: Prepare solution at time of use. Solution expires after one use.

#	Action	Amount	Chemical/Reagent	Details
1	Add	90ml	Distilled/DI Water	Into a chemically cleaned container or new/unused plasticware.
2	Add	10ml	Muci-10 Plus™	Mix thoroughly.

STAINING PROCEDURE:

 Color coordinated steps denote stain baths that can be reused during autostainer configuration.

#	Action	With	Heat °C	Time		Details
				Mins	Secs	
1	Deparaffinize	Xylene or Substitute, 2 changes	--	5	--	5 minutes each change or as required if using a xylene substitute.
2	Rinse	Absolute Alcohol, 3 changes	--	1	--	1 minute each change or as required if using graded alcohols.
3	Rinse	Running Tap Water	--	1	--	
4	Immerse	Modified Mayer's Hematoxylin	--	5	--	Once complete, rinse in running tap water (1 minute) and continue.
5	Immerse	Muci-10 Plus™ Working Solution	--	10	--	If deeper color is desired, extend staining time. Once complete, rinse in running tap water (5 seconds) and continue.
6	Immerse	Metanil Yellow Stain	--	--	15-60	Once complete, rinse in running tap water (5 seconds) and continue.
7	Dehydrate	Absolute Alcohol, 3 changes	--	1	--	1 minute each change.
8	Clear	Xylene or Substitute, 3 changes	--	1	--	1 minute each change or as required if using a xylene substitute.
9	Coverslip	Permanent Mounting Media	--	--	--	

1. Sheehan DC Hrapchak BB: Theory and Practice of Histotechnology, 1980, 168 – 169
2. With modifications by AMTS R&D Department, 1979-2019.

CE MARKINGS AND DESIGNATIONS:

REF	Catalogue Number		Temperature Limitation		Manufacturer	American MasterTech Scientific 1330 Thurman St. Lodi, CA 95240 USA Tel 800 860 4073 Fax 209 368 4136
LOT	Batch Code		Use By	EC REP	Representative	Emergo Europe Prinsessegracht 20 2514 AP The Hague The Netherlands
IVD	In Vitro Diagnostic Medical Device		Consult Instructions Prior to Use	 GHS08	Health Hazard	CE

CONTACT INFORMATION:

American MasterTech Scientific
 1330 Thurman St.
 Lodi, CA 95240 USA
 Tel 800 860 4073
 Fax 209 368 4136
www.americanmastertech.com

StatLab
 2090 Commerce Drive
 McKinney, TX 75069 USA
 Tel 800 442 3573
 Fax 972 436 1369
www.statlab.com

1. Sheehan DC Hrapchak BB: Theory and Practice of Histotechnology, 1980, 168 – 169
2. With modifications by AMTS R&D Department, 1979-2019.

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