Normal Human Colon

Nuclear Fast Red Stain Procedure

0:05

100ml Item#: STNFR100	Liter Item#: STNFRLT	Pint Item#: STNFRPT
Gallon Item#: STNFRGAL	5 Gallon Cube Item#: STNFR5GAL	

Control Slide(s) | Item#

As required for the MasterTech Stain Kits listed below

Required Components (Not Included)

As required for the Alcian Blue pH 2.5 Stain Procedure

PRINCIPLE AND RESULTS: This stain is intended for use by laboratory professionals to stain routinely prepared paraffin embedded tissue specimens (in vitro) to identify nuclei. Nuclei stain red, cytoplasm stains pink, background tissue stains pink to red. No filtration is required when using MasterTech Nuclear Fast Red Stain. MasterTech Nuclear Fast Red Stain acts a counter stain in the following MasterTech Stain Kits: Alcian Blue Stain Kit - pH 1.0, Alcian Blue Stain Kit - pH 2.5, Fontana Masson Stain Kit, Modified Grimelius Microwave Stain Kit, Iron Stain Kit, Jones Basement Membrane Reticulum Kit, Chandler's Precision Reticulum Stain Kit, and Von Kossa Stain Kit.

SPECIMEN CRITERIA: Appropriately fixed, paraffin-embedded, 4-5µm tissue section.

STORAGE AND USAGE NOTES: Store/Use according to the temperature and expiration on the label.

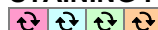
PRECAUTIONS: For use by laboratory professionals. See SDS for complete warnings, precautions, hazard and precautionary statements, and disposal information.

ALCIAN BLUE PH 2.5 STAIN PROCEDURE

3% ACETIC ACID PREPARATION: Prepare solution at time of use. Solution expires after one use.

#	Action	Amount	Chemical/Reagent	Details
1	Add	97ml	Distilled Water	Into a chemically cleaned container or new/unused plasticware.
2	Add	3ml	Glacial Acetic Acid	Mix thoroughly.

STAINING PROCEDURE:

 Color coordinated steps denote stain baths that can be reused during autostainer configuration.

#	Action	With	Heat °C	Time		Details
				Mins	Secs	
1	Deparaffinize	Xylene or Substitute, 2 changes	--	5	--	5 minutes each change or as required if using a xylene substitute.
2	Rinse	Absolute Alcohol, 3 changes	--	1	--	1 minute each change or as required if using graded alcohols.
3	Rinse	Running Tap Water	--	1	--	
4	Immerse	3% Acetic Acid	--	3	--	Without rinsing, continue to next step.
5	Immerse	Alcian Blue Stain pH 2.5	--	30	--	Once complete, rinse in running tap water (1 minute) and continue.
6	Immerse	Nuclear Fast Red Stain	--	5	--	Once complete, rinse in running tap water (1 minute) and continue.
7	Dehydrate	Absolute Alcohol, 3 changes	--	1	--	1 minute each change.
8	Clear	Xylene or Substitute, 3 changes	--	1	--	1 minute each change or as required if using a xylene substitute.
9	Coverslip	Permanent Mounting Media	--	--	--	

1. Sheehan DC Hrapchak BB: Theory and Practice of Histotechnology, 1980, 172 – 173
2. With modifications by AMTS R&D Department, 1979-2019.

CE MARKINGS AND DESIGNATIONS:

REF	Catalogue Number		Temperature Limitation		Manufacturer	American MasterTech Scientific 1330 Thurman St. Lodi, CA 95240 USA Tel 800 860 4073 Fax 209 368 4136
LOT	Batch Code		Use By	EC REP	Representative	Emergo Europe Prinsessegracht 20 2514 AP The Hague The Netherlands
IVD	In Vitro Diagnostic Medical Device		Consult Instructions Prior to Use	 GHS08	Health Hazard	CE

CONTACT INFORMATION:

American MasterTech Scientific
 1330 Thurman St.
 Lodi, CA 95240 USA
 Tel 800 860 4073
 Fax 209 368 4136
www.americanmastertech.com

StatLab
 2090 Commerce Drive
 McKinney, TX 75069 USA
 Tel 800 442 3573
 Fax 972 436 1369
www.statlab.com

1. Sheehan DC Hrapchak BB: Theory and Practice of Histotechnology, 1980, 172 – 173
2. With modifications by AMTS R&D Department, 1979-2019.

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