Human Gastric Tissue

Quick*Stain Stain Procedure

0:02

100ml Item #: N/A

Liter Item#: N/A

Pint Item #: STQUIPT

Gallon Item#: STQUIGAL

Control Slide(s) | **Item#**

Helicobacter Pylori | CSH0125P

PRINCIPLE AND RESULTS: This one-step H&E type stain is intended for use by laboratory professionals to stain routinely prepared paraffin embedded tissue specimens or frozen sections (in vitro) to identify *Helicobacter pylori*. *Helicobacter pylori* stains light blue to dark blue, cytoplasm pink, and nuclei blue to dark blue.

SPECIMEN CRITERIA: Appropriately fixed, paraffin-embedded, 4-5µm tissue section or 5-10µm frozen sections.

STORAGE AND USAGE NOTES: Store/Use each component according to the temperature and expiration on the label.

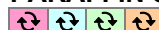
PRECAUTIONS: For use by laboratory professionals. See SDS for complete warnings, precautions, hazard and precautionary statements, and disposal information

FROZEN SECTION STAINING PROCEDURE:

 Color coordinated steps denote stain baths that can be reused during autostainer configuration.

#	Action	With	Heat °C	Time		Details
				Mins	Secs	
1	Immerse	Quick*Stain	--	1-5	--	Or flood on slide rack to desired intensity.
2	Rinse	Running Tap Water	--	--	10-15	Until any remaining stain runs off.
3	Choose mounting method.					
3a	Non Permanent Mount					
1	Coverslip	Water	--	--	--	Review immediately.
3b	Permanent Mount					
1	Air Dry	On Benchtop, or in Empty Bath	--	1-2	--	Until section is completely dry.
2	Clear	Xylene or Substitute	--	--	1	5-10 dips for 1 second each.
3	Coverslip	Permanent Mounting Media	--	--	--	

PARAFFIN SECTION STAINING PROCEDURE:

 Color coordinated steps denote stain baths that can be reused during autostainer configuration.

#	Action	With	Heat °C	Time		Details
				Mins	Secs	
1	Deparaffinize	Xylene or Substitute, 2 changes	--	5	--	5 minutes each change or as required if using a xylene substitute.
2	Rinse	Absolute Alcohol, 3 changes	--	1	--	1 minute each change or as required if using graded alcohols.
3	Rinse	Running Tap Water	--	1	--	
4	Immerse	Quick*Stain	--	2-10	--	Or flood on slide rack to desired intensity. Once complete, rinse in running tap water (10-15 seconds) and continue.
5	Air Dry	On Benchtop, or in Empty Bath	--	1-2	--	Until section is completely dry.
6	Clear	Xylene or Substitute	--	--	1	5-10 dips for 1 second each.
7	Coverslip	Permanent Mounting Media	--	--	--	

1. Lillie RD Fullmer HM: Histopathology Technic and Practical Histochemistry; McGraw-Hill Company, New York; Ed 4, 1976.
2. With modifications by AMTS R&D Department, 1979-2019.

CE MARKINGS AND DESIGNATIONS:

REF	Catalogue Number		Temperature Limitation		Manufacturer	American MasterTech Scientific 1330 Thurman St. Lodi, CA 95240 USA Tel 800 860 4073 Fax 209 368 4136
LOT	Batch Code		Use By	EC REP	Representative	Emergo Europe Prinsessegracht 20 2514 AP The Hague The Netherlands
IVD	In Vitro Diagnostic Medical Device		Consult Instructions Prior to Use	CE		
 GHS02	Flammable	 GHS08	Health Hazard			

CONTACT INFORMATION:

American MasterTech Scientific
 1330 Thurman St.
 Lodi, CA 95240 USA
 Tel 800 860 4073
 Fax 209 368 4136
www.americanmastertech.com

StatLab
 2090 Commerce Drive
 McKinney, TX 75069 USA
 Tel 800 442 3573
 Fax 972 436 1369
www.statlab.com

1. Lillie RD Fullmer HM: Histopathology Technic and Practical Histochemistry; McGraw-Hill Company, New York; Ed 4, 1976.
2. With modifications by AMTS R&D Department, 1979-2019.

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