Human Gastric Tissue

1% Toluidine Blue O Stain Procedure

0:02

100ml Item#: N/A

Liter Item#: STTBOLT

Pint Item#: STTBOPT

Gallon Item#: STTBOGAL

Control Slide(s)
Item#

Helicobacter pylori CSH0125P

Mast Cell CSM0125P

PRINCIPLE AND RESULTS: This stain is intended for use by laboratory professionals to stain routinely prepared paraffin embedded tissue specimens (in vitro) to identify *Helicobacter pylori*, orthochromatic and metachromatic cells and tissues. *Helicobacter pylori* and Nuclei are stained blue, cytoplasm pink, and other tissue metachromatic.

SPECIMEN CRITERIA: Appropriately fixed, paraffin-embedded, 4-5µm tissue section.

STORAGE AND USAGE NOTES: Store/Use component according to the temperature and expiration on the label.

PRECAUTIONS: For use by laboratory professionals. See SDS for complete warnings, precautions, hazard and precautionary statements, and disposal information.

STAINING PROCEDURE: Do not dehydrate 1% Toluidine Blue O stained sections in alcohol; alcohol will remove the stain.

 Color coordinated steps denote stain baths that can be reused during autostainer configuration.

#	Action	With	Heat °C	Time		Details
				Mins	Secs	
1	Deparaffinize	Xylene or Substitute, 2 changes	--	5	--	5 minutes each change or as required if using a xylene substitute.
2	Rinse	Absolute Alcohol, 3 changes	--	1	--	1 minute each change or as required if using graded alcohols.
3	Rinse	Running DI Water	--	1	--	
4	Immerse	1% Toluidine Blue O Stain	--	1-2	--	Once complete, rinse in running DI water (1 minute) and continue.
5	Choose mounting method below.					

5a Aqueous Mount

1	Coverslip	Aqueous Mounting Media	--	--	--	Coverslip Immediately!
---	-----------	------------------------	----	----	----	------------------------

5b Permanent Mount

1	Air Dry	On Benchtop, or in Empty Bath	--	1-2	--	Until section is completely dry.
2	Clear	Xylene or Substitute	--	--	1	Fresh dips. 5-10 dips for 1 second each.
3	Coverslip	Permanent Mounting Media	--	--	--	

1. Sheehan DC Hrapchak BB: Theory and Practice of Histotechnology, 1980, 149.
2. With modifications by AMTS R&D Department, 1979-2019.

CE MARKINGS AND DESIGNATIONS:

REF	Catalogue Number		Temperature Limitation		Manufacturer	American MasterTech Scientific 1330 Thurman St. Lodi, CA 95240 USA Tel 800 860 4073 Fax 209 368 4136
LOT	Batch Code		Use By	EC REP	Representative	Emergo Europe Prinsessegracht 20 2514 AP The Hague The Netherlands
IVD	In Vitro Diagnostic Medical Device		Consult Instructions Prior to Use	CE		

CONTACT INFORMATION:
American MasterTech Scientific

1330 Thurman St.
 Lodi, CA 95240 USA
 Tel 800 860 4073
 Fax 209 368 4136
www.americanmastertech.com

StatLab

2090 Commerce Drive
 McKinney, TX 75069 USA
 Tel 800 442 3573
 Fax 972 436 1369
www.statlab.com

1. Sheehan DC Hrapchak BB: Theory and Practice of Histotechnology, 1980, 149.
2. With modifications by AMTS R&D Department, 1979-2019.

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