Human Aorta

Van Gieson's Stain Procedure

100ml Kit Item #: STVGI

Liter Kit Item#: STVGILT

Pint Kit Item #: STVGIP

Gallon Kit Item#: STVGIGAL

Control Slide(s)
Item#

Elastic

CSA0825P

Required Components (Not Included)

5% Alcoholic Hematoxylin

5% Sodium Thiosulfate

10% Ferric Chloride

Universal Iodine Solution™

PRINCIPLE AND RESULTS: This stain is intended for use by laboratory professionals to stain routinely prepared paraffin embedded tissue specimens (in vitro) to identify collagen, muscle, and red blood cells when used in the Verhoeff-Van Gieson stain procedure. Van Gieson's stains collagen, muscle, and red blood cells yellow.

SPECIMEN CRITERIA: Appropriately fixed, paraffin-embedded, 4-5µm tissue section.

STORAGE AND USAGE NOTES: Store/Use each component according to the temperature and expiration on the label.

PRECAUTIONS: For use by laboratory professionals. See SDS for complete warnings, precautions, hazard and precautionary statements, and disposal information.

VERHOEFF'S ELASTIC STAIN PREPARATION: Prepare solution at time of use. Solution expires after one use.

#	Action	Amount	Chemical/Reagent	Details
1	Add	40ml	5% Alcoholic Hematoxylin	Into a chemically cleaned container or new/unused plasticware.
2	Add	20ml	10% Ferric Chloride	Mix thoroughly
3	Add	20ml	Universal Iodine Solution™	Mix thoroughly.
4	Filter	--	Paper Towel	Filter into a chemically cleaned container or new/unused plasticware.

2% FERRIC CHLORIDE PREPARATION: Prepare solution at time of use. Solution expires after one use.

#	Action	Amount	Chemical/Reagent	Details
1	Add	40ml	Distilled/DI Water	Into a chemically cleaned container or new/unused plasticware.
2	Add	10ml	10% Ferric Chloride	Mix thoroughly

STAINING PROCEDURE:

 Color coordinated steps denote stain baths that can be reused during autostainer configuration.

#	Action	With	Heat °C	Time		Details
				Mins	Secs	
1	Deparaffinize	Xylene or Substitute, 2 changes	--	5	--	5 minutes each change or as required if using a xylene substitute.
2	Rinse	Absolute Alcohol, 3 changes	--	1	--	1 minute each change or as required if using graded alcohols.
3	Rinse	Running Tap Water	--	1	--	
4	Immerse	Verhoeff's Elastic Stain	--	15	--	Once complete, rinse in running tap water (1 minute) and continue.
5	Immerse	2% Ferric Chloride	--	1-5	--	Differentiate until elastic fibers are black (use microscope). Once complete, rinse in running tap water (1 minute) and continue.
6	Immerse	5% Sodium Thiosulfate	--	1	--	Once complete, rinse in running tap water (1-2 minutes) and continue.
7	Immerse	Van Gieson's Stain	--	--	30	
8	Dehydrate	Absolute Alcohol, 3 changes	--	1	--	1 minute each change.
9	Clear	Xylene or Substitute, 3 changes	--	1	--	1 minute each change or as required if using a xylene substitute.
10	Coverslip	Permanent Mounting Media	--	--	--	See storage and usage notes below.

***NOTE:** Some permanent Mounting Mediums may cause fading over time with picric acid based stains. While MicroMount™ or PermaSlip™ is recommended for optimal staining preservation, many other mounting mediums may also provide successful results. Be sure to test your preferred Mounting Medium with this stain if not using one of the recommended products.

- Carson FL Hladik C: Histotechnology, A Self-Instructional Text; 3rd Edition, 2009, 168-170.
- Sheehan DC Hrapchak BB: Theory and Practice of Histotechnology; 1980, 196-197.
- With modifications by AMTS R&D Department, 1979-2019.

CE MARKINGS AND DESIGNATIONS:

REF	Catalogue Number		Temperature Limitation		Manufacturer	American MasterTech Scientific 1330 Thurman St. Lodi, CA 95240 USA Tel 800 860 4073 Fax 209 368 4136
LOT	Batch Code		Use By	EC REP	Representative	Emergo Europe Prinsessegracht 20 2514 AP The Hague The Netherlands
IVD	In Vitro Diagnostic Medical Device		Consult Instructions Prior to Use	CE		

CONTACT INFORMATION:
American MasterTech Scientific

1330 Thurman St.
 Lodi, CA 95240 USA
 Tel 800 860 4073
 Fax 209 368 4136
www.americanmastertech.com

StatLab

2090 Commerce Drive
 McKinney, TX 75069 USA
 Tel 800 442 3573
 Fax 972 436 1369
www.statlab.com

1. Carson FL Hladik C: Histotechnology, A Self-Instructional Text; 3rd Edition, 2009, 168-170.
2. Sheehan DC Hrapchak BB: Theory and Practice of Histotechnology; 1980, 196-197.
3. With modifications by AMTS R&D Department, 1979-2019.

MULTILANGUAGE PROCEDURE

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COMPOSANTS REQUIS (NON INCLUS): 5% Alcoholic Hematoxylin, 10% Ferric Chloride, Universal Iodine Solution™, 5% Sodium Thiosulfate

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