

Instructions for Use

H. pylori Control Slides

CATALOG NUMBER	DESCRIPTION	UNIT OF MEASUREMENT
CS033-25	<i>H. pylori</i> Control Slides, Non-adhesion, 1 stained/24 unstained	1 set
CS033-PS-25	<i>H. pylori</i> Control Slides, Adhesion, 1 stained/24 unstained	1 set
CS033-PS-100	<i>H. pylori</i> Control Slides, Adhesion, 1 stained/99 unstained	1 set

INTENDED USE

Qualitative control for *Helicobacter pylori*.

Please contact productsupport@statlab.com with any additional questions.

TISSUE


Known Positive Tissue

FIXATION

10% Buffered Formalin

TECHNICAL INFORMATION

Paraffin sections cut at 5 microns

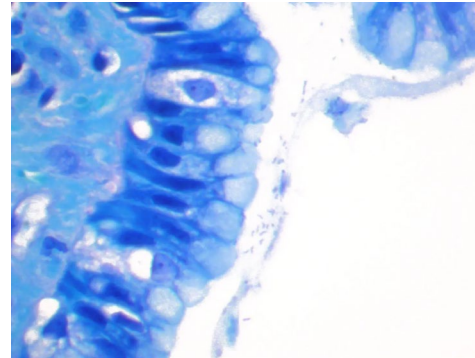

USE WITH MASTERTECH HELICOSTAT STAIN KIT

Kit Size	Part Number
100 mL	KTHST
Pint	KTHSTPT
Liter	KTHSTLT

MASTERTECH HELICOSTAT STAIN PROCEDURE

1. Deparaffinize and hydrate to running DI water.
2. Immerse in Periodic Acid Solution for 10 minutes.
Do not reuse solution.
3. Rinse in running tap water for 1 minute.
4. Immerse in Sodium Metabisulfite Solution for 5 minutes.
5. Rinse in running tap water for 1 minute.
6. Immerse in Alcian Yellow Solution for 5-10 minutes.
7. Rinse in running tap water for 1 minute.
8. Immerse in Toluidine Blue Working Solution for 3-5 minutes.
Do not reuse solution.
9. Rinse in running tap water for 1 minute.
10. Air dry on bench top or empty bath 5-10 minutes until section is completely dry.
11. Clear in Xylene 5-10 dips for 1 second each.
12. Coverslip with Acrymount Plus or any other acceptable mounting medium.

RESULTS

Helicobacter pylori – Blue

Background – Shades of blue

Mucin – Yellow

This protocol is a recommendation only. Modifications may be necessary.



StatLab Medical Products
2090 Commerce Drive, McKinney, TX 75069
StatLab.com Phone: +1 800-442-3573

