

This procedure is available in kit form from:
Poly Scientific R&D Corp.
70 Cleveland Avenue • Bay Shore, NY 11706
(631) 586-0400

cat# cy015

MAY GRUNWALD GIEMSA STAIN PROCEDURE
FOR ASPIRATION BIOPSY SMEARS

SOLUTIONS:

1. Jenner Working Solution
2. Giemsa Stock Solution

Working Giemsa Solution:

Giemsa Stock.....10 mL
Distilled Water.....90 mL

STAINING PROCEDURE:

1. Stain in Jenner Working Solution for 5 minutes.
2. Wash in running water for 1 minute.
3. Stain in Working Giemsa Solution for 15 minutes.
4. Wash in running water for 1 - 2 minutes.
5. Air dry. No mounting necessary.

RESULTS:

Nuclei.....Blue
Cytoplasm.....Pink to Rose
Bacteria.....Blue