

Fungi-Tape

Manufacturer/Supplier:

Scientific Device Laboratory
411 Jarvis Ave, Des Plaines, IL 60018 USA
General and Technical Information
Phone Number: 847-803-9495
Website: www.scientificdevice.com

Intended Use:


Fungi-Tape is precut cover tape for making Mycology slide preparations.

Summary and Explanation:

The product is an adhesive tape that is pliable enough to make a tape loop and then sample from a growing fungus culture. Fungi-Tape can be used with MycoPerm Blue and a microscope slide to create a long lasting mount of fungi.

Instructions/ Procedure:

- Pull tape and backing out of box.
- Fold backing around flap at top of box.
- Pull backing at end and each tape-cover slip will separate from backing.
- Only hold tape at the ends and make a loop.
- Gently apply (touch) tape to growing fungal culture.

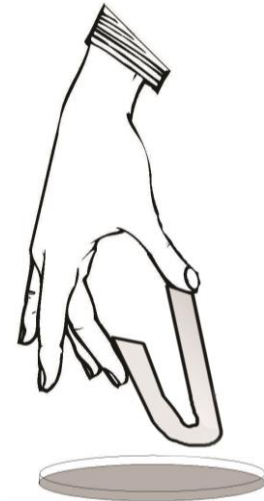

NOTE: THIS PROCEDURE SHOULD BE PERFORMED WITH PROPER MICROBIOLOGY TECHNIQUES AND UNDER SAFETY HOOD WITH GLOVES ON.

- Place tape onto microscope slide that has a drop of MycoPerm-Blue mounting fluid on it.
- Release edges of tape and smooth over the microscope slide to get even adhesiveness.

Components:

One roll of Fungi-Tape packaged in dispenser box has approximately 360 pieces of tape.

Storage:

Store at room temperature (15°C to 30°)

Limitations:

Make sure tape is laid down gently and evenly on microscope slide to prevent air bubbles from forming.




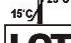




Safety:

For *In vitro* diagnostic use information. Good general microbiology techniques should be followed. Gloves should be worn when performing touch prep using Fungi-Tape and growing fungal culture. Fungal cultures should be worked with under safety hood.

Related Product Information:

- Catalog # 745: Fungi-Tape (1 roll approximately 360 pieces of tape)
- Catalog # 745-1: Mycology Mounting Kit – Blue (one 12ml bottle MycoPerm Blue & 1 roll Fungi-Tape)

SYMBOL LEGEND

	Catalog Number
	In Vitro Diagnostic Medical Device
	Consult Instructions for Use (IFU)
	Temperature Limitations (Storage Temp.)
	Batch Code (Lot Number)
	Use By (Expiration Date)
	Quantity Included
	Dimensions

Supported Products Page

This Technical Insert is used for the following products:

SDL Prod ID	Description			
507	745	.	Fungi-Tape	1 roll/pkg
508	745	-1	Mycology Mounting Kit-BLUE	1 Fungitape, 12ml MycopermBLUE/pkg

Revision History

CR NUMBER	REVISION
0210-010	00
1017-003	01
0420-001	02
1222-001	03